


Instructions for your safety syringe


Please read these instructions in full or if you would like to watch the instructional video, scan the QR code



Before use

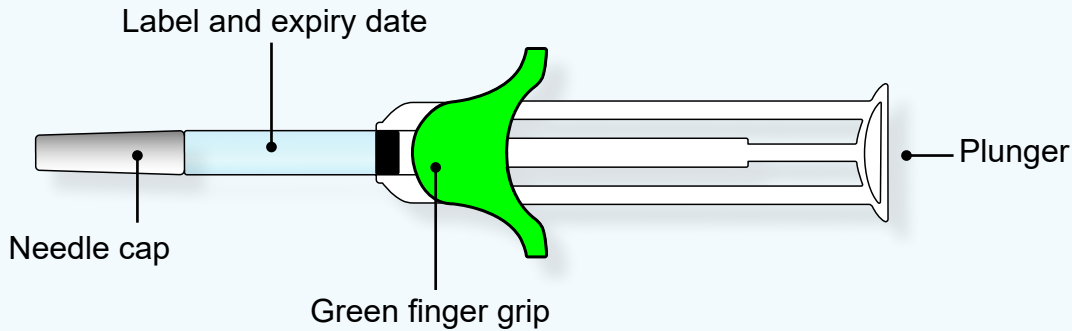

After use

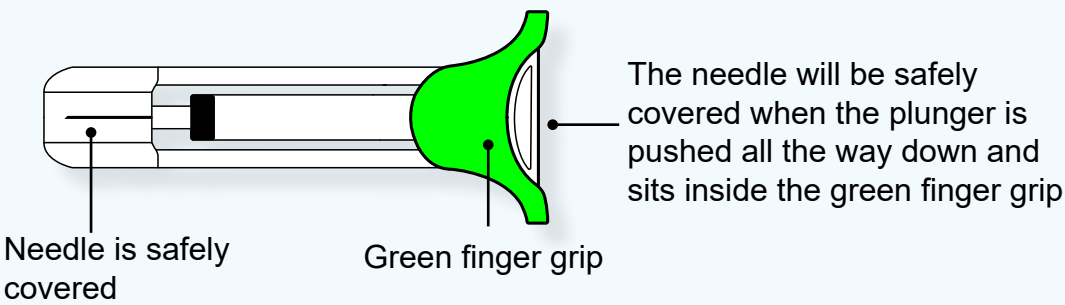

B Preparation

1		Wash your hands with soap and water
2		Choose your injection site Do not inject into areas that are red, hard, bruised, or tender.
3		Wipe the skin at the injection site with an alcohol swab.

C Injecting the medicine

			<div><div></div><div>Warning</div></div> <div></div> <div></div>	
1 Pull the needle cap straight off. Throw needle cap in bin.	2 Insert the needle into the skin (45° to 90°).	3 Push the plunger all the way down until it fully sits inside the green finger grip.	4 Make sure the plunger is all the way down. If plunger is not all the way down, the needle is not safely covered.	5 Dispose of the safety syringe and needle cap in an approved sharps container.