ENGLISH:

Heel incision device. Instructions for use.

The intended use: Heel incision safety lancets are single use devices used to collect a capillary blood sample from the heel of newborn infants. Only single use disposable blood lancets should be used for assisted blood draws by healthcare providers or at healthcare provision sites.

Wash hands thoroughly before and after using the device. In accordance with your institution's guidelines, use personal protective equipment, gently warm the infant's heel and clean/disinfect the outer side of the heel where you will perform the incision. Remove the sterility tab by pulling straight off (1). There are two ways to grip the device. The preferred position is with thumb and finger gripping on the wider side. Alternatively, you may grip with your thumb against the narrow side (2). Using the alignment guide (A), align the blade slot with the incision site (3), then activate the button. Blade will permanently retract after use. **Dispose of in a suitable sharps container.** To express blood, gently apply pressure at the top and bottom of the incision site.

If any serious incident occurs with the use of this device please report to Owen Mumford Limited and your local competent authority.

The cap is the sterile barrier, do not use if cap previously removed. **For use only on a single patient. Discard the entire device after use.** Warning: Not intended for more than one use. Do not use on more than one patient. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested. The lancet is designed with a spring to automatically retract the needle into the body of the device after use to prevent injury. Risk of injury: Before using the lancing device on infants ensure the use of appropriate penetration depth in accordance with your institutions guidelines.

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